# SharpSafety™

## Hinged Lid Sharps Containers

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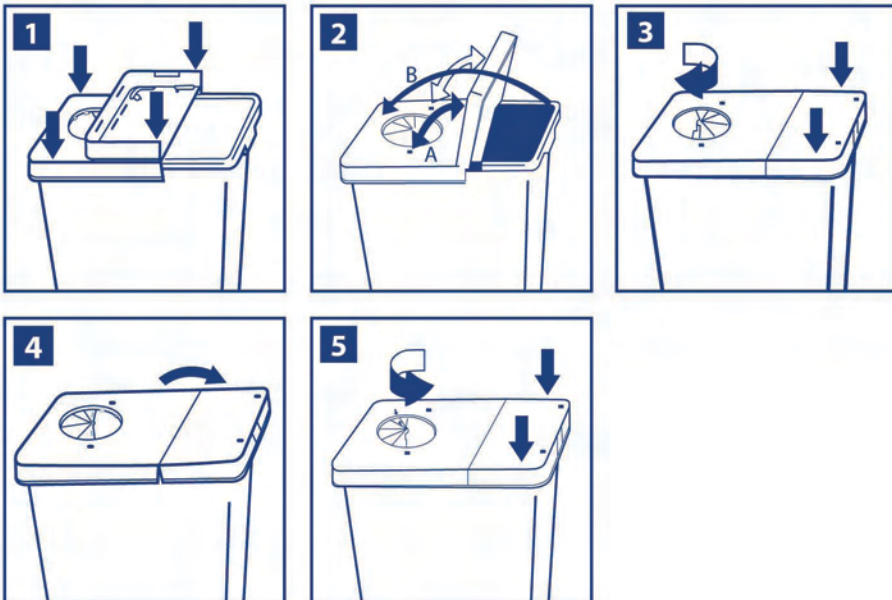
**Assembly**

1. Snap lid on container with hinged lid open. Listen for snaps. Please check that the container has suffered no shipping damage before placing into use.
2. To use hinged lid – (A) Open and secure lid by locking lid's lugs in place or (B) Be sure lid is at rest in a fully open position.

**Stabilization**

To maintain container upright and stable, install container in appropriate Covidien hardware. Check with your Covidien account representative for product catalog for cross-references.

**Usage Instructions**

To dispose of waste

Use one hand to drop waste in container then temporarily close hinged lid. Drop one sharp in at a time.

3. To use rotor opening – Close hinged Lid. Open rotor by turning clockwise until opening is the desired size. Use one hand to drop one sharp in at a time. Rotor may be closed temporarily when not in use to prevent accidental contact with container contents.

**Filling/no overfilling**

Do not fill container above recommended fill line or when three quarters full if fill line is not present.

**To temporarily close door**

4. Hinged lid may be closed temporarily when not in use by lightly closing door. If using a foot pedal cart with this container, follow foot pedal cart instructions for use.

**For final closure and locking**

5. To lock container for final closure, lock hinged lid by pressing down securely on all edges of the lid. Listen for snaps. To lock rotor – Turn rotor counterclockwise to the locking position – Lining up arrows.

The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

**Carrying/lifting instructions**

Never hold or carry used container below recommended fill line. If handles are present, use both handles on container when moving or lifting for disposal. Use additional caution with containers that do not have handles.

**Precautions/warnings**

- Always keep used container in upright position.
- Never put hands into container opening.
- Use the proper size container for the items and volume being disposed.
- Do not place anything on top of container.
- Do not strike, tap, shake or manipulate container for any reason, i.e. to settle contents.
- Never hold or carry used container below recommended fill line.
- Use both handles on container when moving or lifting for disposal.
- Never hold or carry container by the lid.
- Do not hold or carry container close to the body.
- Never attempt to disassemble container. Exposure may occur to biohazardous materials and cause percutaneous or other injuries.
- Use container with appropriate Covidien hardware that fits with the Covidien container of choice. See product catalog for cross-reference.
- For single use only. Dispose of used container according to facility protocol.

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